


Easy to use:

Step by step inhalation with the Easyhaler®


1

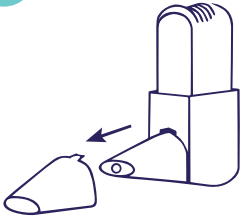

- Remove the safety cap from the mouthpiece

Shake


2

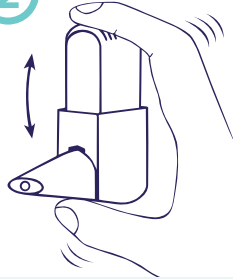

- Shake the Easyhaler® vigorously 3 to 5 times

Click


3

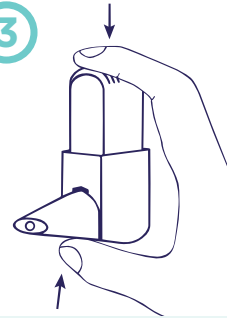

- Hold the inhaler upright
- Squeeze firmly between your thumb at the base and your index finger at the top, until you hear a click
- Then let it click back to its original position

Inhale

4

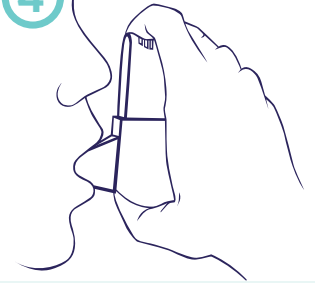

- Breathe out normally (but NOT into the Easyhaler®)
- Close your lips tightly around the Easyhaler®
- Breathe steadily and deeply through the Easyhaler®
- Remove the inhaler and hold your breath for 5 seconds
- Then breathe out next to the device
- Replace the safety cap onto the mouthpiece


If you have been prescribed more than one dose per application, repeat steps 2 to 4.

To reduce the risk of a fungal infection, rinse your mouth and gargle with water but DO NOT swallow.

The counter

The Easyhaler® has a counter showing how many single doses are left in the inhaler.


- The counter moves after every 5 applications
- When the numbers turn red: there are only 20 single doses left
- When the counter reaches 0, the Easyhaler® has to be replaced

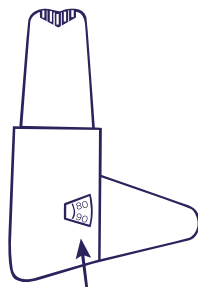

If you are using a protective case

Do NOT remove the Easyhaler® from the protective case for inhaling.

- Open the protective case
- Remove the cap from the mouthpiece
- Inhale as described in steps 2 to 4
- Replace the cap on the mouthpiece
- Close the protective case

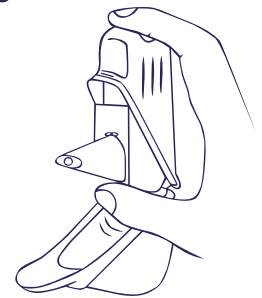

Troubleshooting


The Easyhaler® design prevents double dosing!

If...

- you accidentally load the Easyhaler®
- you press the Easyhaler® more than once
- you blow into the inhaler

Then...

- tap the mouthpiece of the Easyhaler® onto a table or your hand

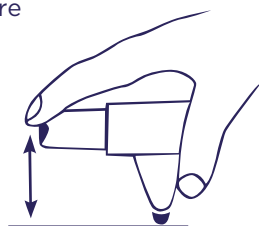

For more information, please visit our website:
www.easyhaler.de



ORION Pharma GmbH

Notkestraße 9 · 22607 Hamburg
Fon 040-899689-0
info-hamburg@orionpharma.com
www.orionpharma.de

 Manufactured in Finland

ORION PHARMA

EASYHALER